

Lactoangin (logotype 1x)

MEDICAL DEVICE

SPRAY FOR MOUTH AND THROAT

PRODUCT DESCRIPTION

Lactoangin spray is a specialistic medical device intended for supportive treatment of upper respiratory tract infections (mouth, throat). **Lactoangin** moistens, soothes and relieves pain.

Lactoangin contains metabolites of lactic acid bacteria of *Lactobacillus* genus, in particular lactic acid, acetic acid, other organic acids, menthol and mint oil, which naturally support treatment of upper respiratory tract infections (mouth, throat).

Lactoangin also facilitates restoration of microbiological balance in mouth and throat, which reduces the risk of caries development, formation of dental plaque and gum inflammation, and eliminates bad breath.

INGREDIENTS

Lactoangin is composed of metabolites from lactic acid bacteria of *Lactobacillus* genus, menthol, mint oil and excipients: glycerol, propylene glycol, natural lemon flavour, potassium sorbate, xanthan gum, sucralose and water.

Lactoangin is used for:

- supportive treatment of mouth and throat infections, including candidiasis,
- soothing throat pain,
- moistening of mouth and throat.

Lactoangin is also used as an aid in states of **microflora balance disorders** of mouth and throat in order to:

- eliminate bad breath,
- prevent caries,
- prevent formation of dental plaque and gum inflammation.

DOSAGE

Children over 3 years of age to 6 years of age: 1 dose of preparation up to 4 times daily or in accordance with doctor recommendation.

Children over 6 years of age and adults: 2-3 doses of preparation up to 4 times daily or in accordance with doctor recommendation.

It can be used in children over 3 years of age. In children under 3 years of age use after consulting the doctor.

In case of omission of the dose, subsequent dose should be administered. Do not take double dose to supplement the omitted dose.

INSTRUCTION FOR USE

1. Shake well before use.
2. Raise the applicator by twisting aside.
3. Before the first use, repeatedly press the pump to fill the applicator and to obtain a proper spraying.
4. Place the applicator in mouth, direct its end to the area of mouth or throat, and then press the pump to release the dose of the preparation. Repeat activity appropriate number of times.

WARNINGS

Do not use if you are hypersensitive to any of the ingredients of the device.

During pregnancy and breast-feeding the preparation should be used after medical consultation.

Do not use after the expiry date stated on the packaging.

Do not use if the immediate packaging is damaged.

Do not use with other locally used products.

Keep out of the sight and reach of children.

If after product using any disturbing symptoms appear, i.e. burning sensation, intensification of pain etc. stop using the product and consult a doctor.

STORAGE CONDITIONS

Store in tight closed immediate packaging, at room temperature (15-25°C), in dry place.

Do not store in a refrigerator, do not freeze.

PACKAGING

The bottle with applicator which contains 30g of preparation, in cardboard box with instruction for use.

PRODUCT SHELF LIFE

2 years



IBSS BIOMED S.A.

Al. Sosnowa 8

30-224 Krakow, Poland

Tel.: + 48 12 37 69 200

Fax: + 48 12 37 69 205

e-mail: bdu@biomed.pl

PREPARED/UPDATED ON

2015-06

IBSS BIOMED S.A. (LOGO)

265 ||||| (Pharma Code)



Manufacturer




Use by date



Batch code



Consult instructions for use

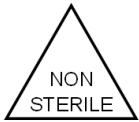

Non-sterile



Temperature limit