

**prOVag gel** (logotype lx)

MEDICAL DEVICE

## **SPECIALIST GEL FOR WOMEN AND CHILDREN FOR PREVENTING AND FOR SUPPORTING TREATMENT OF INFECTIONS OF INTIMATE AREAS**

IBSS BIOMED S.A. (logo lx)

### **PRODUCT DESCRIPTION**

prOVag gel is a specialist medical device intended for supporting treatment of bacterial and fungal infections of external intimate areas.

Components of prOVag gel maintain the physiological, acidic pH and protect the microbiological balance of intimate areas. prOVag gel has antifungal properties and limits the growth of *Candida albicans* and *Candida glabrata*. It also acts antibacterially and protects against microorganisms the most commonly responsible for vaginitis, such as: *Gardnerella vaginalis*, *Escherichia coli*, *Prevotella bivia*, *Streptococcus agalactiae*.

prOVag gel is recommended for women and children over 1. year of age and older that are susceptible to infections of intimate areas to support treatment and alleviate infection symptoms. prOVag gel reduces burning sensation and discomfort, alleviates irritations and other symptoms associated with inflammatory states of intimate areas. It has moisturising and caring properties.

prOVag gel is also recommended in daily prevention of bacterial and fungal infections.

*Hypoallergenic product recommended for people with sensitive skin, prone to allergies, atopy, over-drying and irritation within the intimate areas.*

*Product dermatologically tested on persons with positive allergy history.*

*Product gynaecologically tested.*

*Manufactured under Good Manufacturing Practice conditions.*

In order to achieve stronger and long-lasting effects simultaneous use of prOVag gel and prOVag dietary food for special medical purposes in the form of oral capsules is recommended.

**prOVag gel has been approved by the Polish Gynaecological Society.**

### **INGREDIENTS**

prOVag gel is composed of *Lactobacillus* lactic acid bacteria metabolites, mainly lactic acid and other organic acids which naturally protect intimate areas against unfavorable microorganisms by maintaining low pH. prOVag gel also contains water and excipients: propylene glycol, glycerin, hydroxypropyl methylcellulose, phenethyl alcohol, caprylyl glycol and fragrance.

*prOVag gel does not contain PEG, parabens and detergents, including: SLS, SLES.*

### **WHEN TO USE PROVAG GEL?**

prOVag gel is recommended for women and children with tendency to dryness of external intimate areas, disorders of the natural microflora and recurring inflammations.

prOVag gel can be used on a daily basis or as a “rescue” in order to:

- support treatment of bacterial and fungal infections of intimate areas
- maintain the balance of microflora of intimate areas

- alleviate symptoms of bacterial and fungal infections, such as: burning sensation, itching and discomfort of the external intimate areas
- maintain the physiological, acidic pH of intimate areas, which inhibits excessive growth of bacteria and fungi

and as a prevention in the states of increased risk of inflammation in intimate areas, especially:

- during and after antibiotic therapy (oral and vaginal)
- when it is difficult to maintain a proper hygiene, e.g. during travel
- when using swimming pool, hot tub, or sauna frequently
- after intimate areas depilation
- during climacterium, after puerperium, and in diabetes.

## **POSOLOGY**

Usually, a single daily application is sufficient. If necessary, prOVag gel can be applied 2–3 times a day.

## **INSTRUCTION FOR USE**

After washing intimate areas, apply a thin layer of prOVag gel around perineum and external genitals area. prOVag gel should be applied from the front to the direction of the rectum.

prOVag gel is comfortable in use, does not leave stains on the underwear and its special consistency ensures pleasant and long-lasting moisturising.

## **WARNINGS**

Do not use if you are hypersensitive to any of the ingredients of the product.

In the case of acute vaginal infections it is recommended to use under medical supervision.

Due to the lack of reports from relevant studies, consult your doctor before using this product during pregnancy and breast-feeding.

Do not use after the expiry date stated on the packaging.

Do not use if the packaging is damaged.

Do not swallow.

Keep out of the sight and reach of children.

A temporary, slight tweaking may occur after applying the product, which is associated with the normal response to decreased pH.

If any alarming symptoms, such as: intensified itching, intensified burning sensation, rash, etc. occur after the application of prOVag gel, stop using it and consult a doctor.

## **STORAGE CONDITIONS**

Store in temperature between 15°C and 25°C in a dry place.

## **PACKAGING**

30 g Airless container, in a cardboard box with instruction for use.

## **PRODUCT SHELF LIFE**

3 years





IBSS BIOMED S.A.  
Al. Sosnowa 8,  
30-224 Krakow, Poland  
Tel: +48 12 37 69 200  
Fax: + 48 12 37 69 205  
e-mail: [bdu@biomed.pl](mailto:bdu@biomed.pl)  
[www.biomed.pl](http://www.biomed.pl)

**PREPARED/UPDATED ON**  
2016-03

IBSS BIOMED S.A. (LOGO)

364 IIIIIII (Pharma Code)



*Manufacturer*



*Use by date*




*Batch code*



*Consult instructions for use*



*Non-sterile*

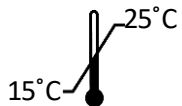

*Temperature limit*